Respironics Inc.  
1001 Murry Ridge Lane  
Murrysville, PA 15668 USA

Respironics Deutschland  
Gewerbestrasse 17  
82211 Herrsching, Germany  
**EC REP**

© 2015 Koninklijke Philips N.V. All rights reserved.

1070335 R06  
LZ 3/23/2015

ComfortGel Blue | A small blue circular icon featuring a white stylized figure reading a book.

**PHILIPS**  
 A black swoosh graphic above the word "RESPIRONICS".  
**RESPIRONICS**

# ComfortGel Blue

EN	001	INSTRUCTIONS FOR USE	SL	086	NAVODILA ZA UPORABO
FR	006	MODE D'EMPLOI	LV	091	LIETOŠANAS INSTRUKCIJAS
IT	011	ISTRUZIONI PER L'USO	LT	096	NAUDÖJIMO INSTRUKCIJOS
DE	016	GEBRAUCHSANWEISUNG	ET	101	KASUTUSJUHISED
ES	021	INSTRUCCIONES DE USO	RO	106	INSTRUCȚIUNI DE UTILIZARE
PT	026	INSTRUÇÕES DE UTILIZAÇÃO	BG	111	ИНСТРУКЦИИ ЗА УПОТРЕБА
PT-BR	031	INSTRUÇÕES DE USO	MK	116	УПАТСТВА ЗА УПОТРЕБА
EL	036	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	HR	121	UPUTE ZA UPOTREBU
DA	041	BRUGSANVISNING	ZH-CN	126	使用说明
NL	046	GEBRUIKSAANWIJZING	ZH-TW	131	使用說明
SV	051	BRUKSANVISNING	RU	136	ИНСТРУКЦИИ ПО ЭКСПЛУАТАЦИИ
FI	056	KÄYTTÖOHJEET	SR	141	UPUTSTVO ZA UPOTREBU
NO	061	BRUKSANVISNING	HE	146	הוראות שימוש
PL	066	INSTRUKCJA STOSOWANIA	TR	151	KULLANIM TALİMATLARI
CS	071	NÁVOD K POUŽITÍ	KO	156	사용 지침
HU	076	HASZNÁLATI ÚTMUTATÓ	AR	161	تعليمات الاستخدام
SK	081	NÁVOD NA POUŽITIE			

# Instructions for Use

## ComfortGel Blue

ENGLISH

### Intended Use

The ComfortGel Blue nasal mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients (>30kg) for whom CPAP or bi-level therapy has been prescribed.

### Notes:

- An exhalation port is built into the ComfortGel Blue mask so a separate exhalation port is not required.
- This mask is not made with natural rubber latex or DEHP.

### Symbols

Warning or Caution

Note

Tip

Consult Instructions  
for Use

Not Made With  
Natural Rubber  
Latex


X1 System One  
Resistance  
Control Setting

### Warnings:

- This mask is not suitable for providing life support ventilation.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port.** **Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.

- If an additional exhalation device is added to the patient circuit, you may need to adjust the pressure level to compensate for the additional leak of the exhalation device.
- The gel mask is not customizable. Do not attempt to heat.
- Hand wash prior to first use. Inspect the mask for damage or wear (cracking ,crazing, tears, cushion damage resulting in gel exposure, etc). Discard and replace any components as necessary.
- Use of a nasal or full face mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. Consult your physician or dentist if symptoms occur.

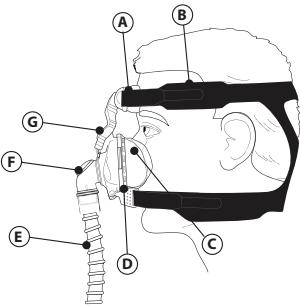 **Figure 1**

A = Forehead Pad  
 B = Headgear Tab  
 C = Gel Cushion with  
     Comfort Flap  
 D = Quick Clip and  
     Faceplate Socket  
 E = Tubing  
 F = Elbow with Exhalation  
     (Do not block or remove)  
 G = StabilitySelector

## Before Use

- Read and understand the instructions.
- Hand wash mask.
- Inspect all parts for damage; replace any parts that are visibly deteriorated (cracking, crazing, tears, etc).
- Clean your/the patient's face.
- Verify that the mask and headgear are the correct size.

## Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask. To disassemble the mask for cleaning, please refer to the **Cushion Removal** later in this document.

1. Hand wash the mask and headgear in warm water with a mild liquid dish washing detergent.

**⚠ Caution:** Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

**⚠ Caution:** Any deviation from these instructions may impact the performance of the product.

2. Rinse thoroughly with water and air dry out of direct sunlight. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.

**⚠ Warning:** Inspect the mask for damage or wear (cracking ,crazing, tears, cushion damage resulting in gel exposure, etc). Discard and replace any components as necessary.

## Institutional Disinfection

For multi-patient use in the hospital/institutional environment, use the Disinfection Guide to reprocess the mask between patients. These instructions can be obtained by visiting us online at [www.philips.com/respironics](http://www.philips.com/respironics) or by contacting Philips Respironics Customer Service at 1-724-387-4000 or Respironics Deutschland at +49 8152 93060.

## Dishwashing Instructions

In addition to hand washing, the mask may be cleaned in the dishwasher once a week.

**⚠ Caution:** Use a mild liquid dish washing detergent only.

1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Separate the cushion and the faceplate and wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.

4. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc). Replace the mask if the cushion becomes hardened.

### System One Resistance Control

This mask uses the System One Resistance Control setting . This symbol may appear on your therapy device. This symbol represents the level of mask resistance compensation. Match the setting to the mask if appropriate.

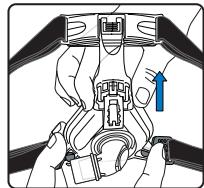
Figure 2

#### Important Notes:

- Verify the System One setting if the mask or mask cushion changes.
- The System One setting is not compatible with masks requiring a separate/additional exhalation device.

If your provider has locked the resistance compensation setting, you can view the setting but cannot change it, and the screen will display a lock symbol. If your provider has disabled the resistance compensation, you will not see this setting.

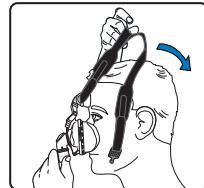
Figure 3

### Taking Off the Headgear

Undo the tabs on the headgear and pull them through the slots in the forehead support and through the slots on the Quick Clips.

To reattach the headgear, thread the top headgear straps through the slots in the forehead support and through the slots on the Quick Clips. Fold the tabs to latch onto the headgear straps.

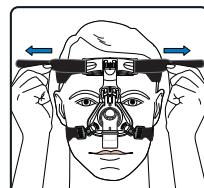
Figure 4

### Achieving the Right Fit

1. Position the StabilitySelector to its uppermost position (Figure 2). Hold the mask cushion over your nose.  
**Note:** You may want to disconnect a Quick Clip before putting on the headgear.
2. Slide the headgear over your head (Figure 3). Reconnect the Quick Clip after you have the headgear in position.
3. Adjust the headgear evenly using the top (Figure 4) and bottom (Figure 5) headgear straps. The mask should rest comfortably against your face.
4. Connect the flexible tubing from your therapy device to the mask elbow. Turn on the airflow.
5. Minimal leaks are acceptable. Small adjustments are all that are needed to eliminate excessive leaks. We suggest the following adjustments:
  - Re-seat the mask by pulling the mask directly away from your face and set it back into place gently (Figure 6).
  - For leaks around the eyes, use the StabilitySelector and adjust downward one click at a time (Figure 7).
  - For leaks around the mouth, a slight adjustment to the lower headgear straps will minimize the leak, do not overtighten.
6. Make any final adjustments lying down.

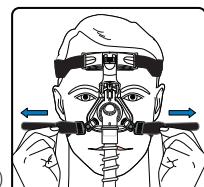
Figure 5

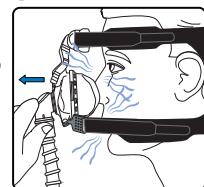
Figure 6

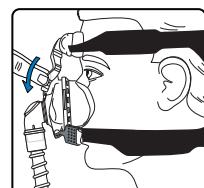
Figure 7


## Comfort Tips

- Re-seat the mask by pulling it directly away from your face and gently setting it back into place.
- Use the StabilitySelector to adjust for leaks or pressure. Some leakage is normal.
- The headgear should fit comfortably and the mask should not dig into your skin. If your skin bulges around the mask or if you see red marks on your face due to tightness, loosen the headgear.

 **Note:** Do not attempt to separate the StabilitySelector from the faceplate rail.

## Cushion Removal and Attachment


1. Remove the gel cushion by detaching the comfort flap with its integrated retaining ring from the faceplate. Do this by unhooking the side and bottom clips of the ring from their connection points on the faceplate and separating the pieces.
2. Clean or discard the gel cushion as necessary.
3. To attach, place the clean or new gel cushion into the faceplate. Then place the comfort flap over the gel cushion and push the retaining ring into the faceplate until it clicks into place.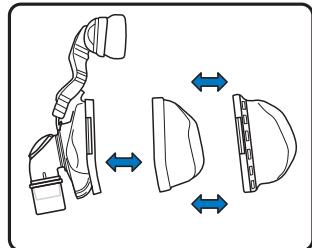

## Specifications

 **Warning:** The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

## Pressure Flow Curve

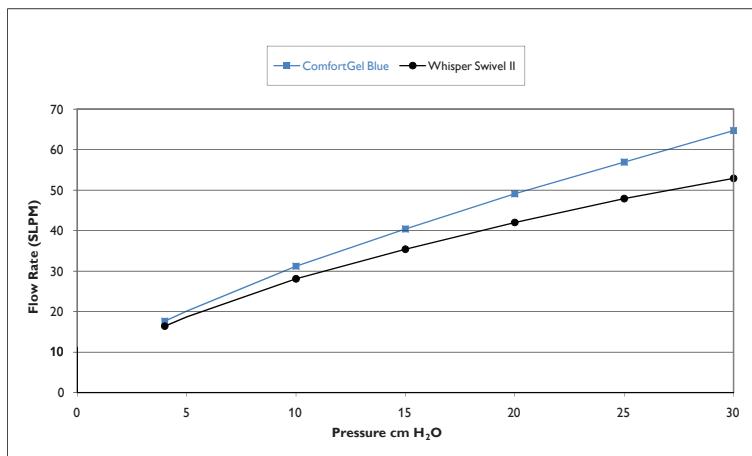## Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0.1 cm H <sub>2</sub> O	0.7 cm H <sub>2</sub> O

## **Deadspace Volume**

Petite: 76 mL  
Small: 97 mL  
Medium: 99 mL  
Large: 118 mL

## **Sound Levels**

A-weighted Sound Power Level	28 dBA
A-weighted Sound Power Pressure Level at 1 m	20 dBA

## **Disposal**

Dispose of in accordance with local regulations.

## **Storage Conditions**

Temperature: -4° to 140° F (-20° to +60° C)  
Relative Humidity: 15% to 95% non-condensing

## **Limited Warranty**

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship. This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA, or Respironics Deutschland GmbH & Co KG, Gewerbestraße 17, 82211 Herrsching, Germany.

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