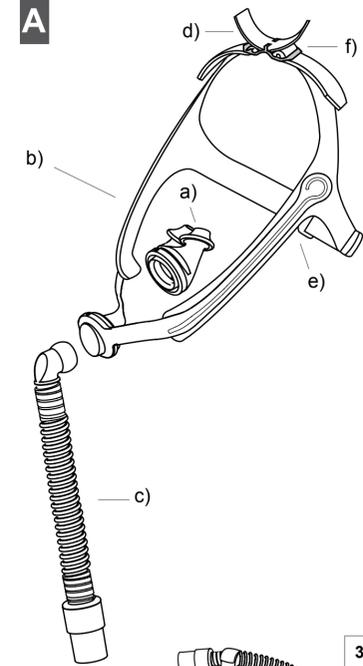
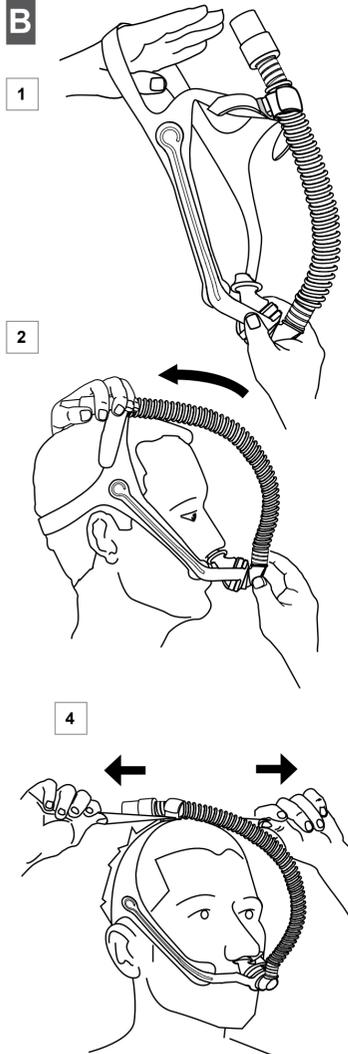
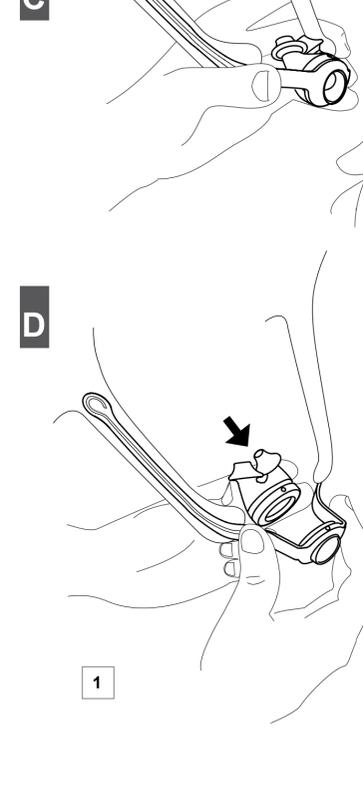
A**B****C****D**

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ENGLISH

NASAL PILLOW MASK

for use by a single patient

Instructions for the patient

1 - APPLICATION

Respiro Primo P Non Vented is a non-vented nasal pillow mask for use by a single patient.
Respiro Primo P Non Vented is designed for use by patients with controlled non-invasive positive pressure ventilation (NPPV).
Respiro Primo P Non Vented is suitable for adult patients (>30 kg).

2 - IMPORTANT NOTES

- The mask must only be used with a ventilator fitted with an active exhalation valve and/or with a respiratory circuit with an exhalation valve.
- The operating pressure range is 4-24 cmH₂O.
- Consult your family doctor or therapist if you experience any discomfort, irritation or an allergic reaction to any of the components.
- If oxygen is used with the ventilator, the oxygen supply must be turned off when the ventilator is not in use.
- Never smoke or use naked flames when using oxygen.
- The mask needs to be cleaned and disinfected. For further details, refer to section 7 - CLEANING THE MASK.
- Check the mask thoroughly before using it for the first time. If you notice any damage, inform your local dealer.
- The appliance must never be used for applications other than that described in this manual.
- Some of the components are small enough to be swallowed by children, so they must never be left unattended. An adult must always be present if the mask is used by a child.

3 - COMPONENTS OF THE MASK (Fig. A)

a) nasal pillow, b) main body, c) tube assembly; d) tube strap; e) rear buckle; f) top buckle.
REMARK: The pack contains three nasal pillows sizes S, M and L.

4 - HOW TO PUT ON THE MASK (Fig. B)

The mask comes fully assembled (size M) and ready for use. Proceed as follows to put on the mask.
Insert the tube assembly (c) in the tube strap (d).
Hold the main body (b) firmly in one hand, keeping the sidepieces upwards. With the other hand, extend the stretch part of the main body (Fig. B1) with the other hand.
Place the main body (b) over your head and insert the nasal pillows (a) into your nostrils (Fig. B2).
Adjust the tension of the rear strap (Fig. B3).
Adjust the tension of the top strap (Fig. B4).
Connect the mask to the patient circuit of the ventilator.

5 - HOW TO REMOVE THE MASK

To remove the mask, pull the nasal pillows (a) out of the nose and lift the main body (b) off your head. You do not need to loosen the buckles.

6 - HOW TO DISASSEMBLE THE MASK (Fig. C)

The mask comes fully assembled (size M) and ready for use. When it needs to be cleaned, proceed as follows to disassemble it.
Remove the elbow from the main body (b) to disconnect the tube assembly (c) from the mask (Fig. C1).
Remove the nasal pillows (a) from the main body (b) (Fig. C2).

7 - CLEANING OF THE MASK

The mask must be washed before each use, so it needs to be disassembled (see 6 - HOW TO DISASSEMBLE THE MASK).

DAILY CLEANING

Wash the nasal pillows (a), the tube assembly (c) and the plastic part of the main body (b) in lukewarm water (about 30 °C) with mild soap. Rinse thoroughly in water and allow to dry naturally.

WEEKLY CLEANING

Wash the nasal pillows (a), the tube assembly (c) and the main body (b), including the fabric part, as instructed under daily cleaning.
The whole mask can be cleaned in a washing machine at 30 °C, but without spin-drying (delicate wash) up to 20 times.

8 - PRECAUTIONS

Do not clean the mask with solutions containing alcohol, aromatic compounds, humidifiers, antibacterial agents or aromatic oils. Such solutions would damage the product and reduce its life.
Prolonged exposure to direct sunlight or heat may cause damage to the mask.
If any visible damage is noted (cracks, tears, etc.), the damaged part must be replaced.
Check carefully to make sure there are no soap deposits or foreign bodies inside the mask.

9 - HOW TO REASSEMBLE THE MASK (Fig. D)

Insert the nasal pillows (a) in the slot in the main body (b). Make sure the two reference points are aligned, then push the nasal pillows as shown in the diagram (Fig. D1).
Insert the elbow of the tube assembly (c), in the slot in main body (b) (Fig. D2).
If you wish, you can secure the tube (c) in the tube strap (d).

10 - TROUBLE-SHOOTING CHART

PROBLEMS	POSSIBLE CAUSES	SOLUTIONS
• No air or insufficient air comes out of the mask	• Flow generator off • Flow generator regulated wrongly • Patient circuit tube not inserted	• Switch on the flow generator • Regulate the flow generator • Connect the tube to the flow generator and mask
• The mask is noisy	• Nasal pillows mounted incorrectly and air leak • You are using the wrong size	• Mount the cushions correctly as instructed • Replace with a more suitable size (sizes available S - M - L)
• The nasal pillows do not fit into the nasal cavity		

11 - DISPOSAL INSTRUCTIONS

The components of the mask are made of the following materials and must be disposed of in accordance with the applicable laws.

COMPONENT	MATERIAL
Nasal pillows	Silicone
Main body	Nylon
Elbow	Polycarbonate
Tube	Metallocene
Connector	Acetalic
Swivel fitting	Polycarbonate
Headgear	Nylon/Polyurethane
Tube strap	Nylon/Polyurethane
Rear buckle	Nylon
Top buckle	Nylon

12 - TECHNICAL DATA

Normative references	CE marking in accordance with European Directive 93/42/EEC, class Ila and subsequent amendments. EN ISO 14971, ISO 5356-1, ISO 10993-1, EN ISO 17510-2
Operating pressure	4 – 24 cmH ₂ O
Temperature range	5 – 40 °C
Dead space (mask and tube assembly)	81 ml

13 - LIMITED WARRANTY

Air Liquide Medical Systems S.r.l. warrants that your product shall be free from defects in material and workmanship for a period of 120 days from the date of purchaser.
If the product fails under conditions of normal use, Air Liquide Medical Systems S.r.l. will repair or replace, at its option the defective product or any of its components.
The manufacturer is liable for the safety, reliability and performance of the apparatus, provided that it is used in accordance with the instructions and for the intended purpose only, and that any repairs are made by the manufacturer or an approved service centre.
Throughout the warranty period keep the original packing.
Warranty claims on effective product must be made by the consumer contacting the dealer.
For further information on your warranty rights contact your dealer.

COPYRIGHT

The information contained in this booklet may not be used for anything other than the intended purpose. This booklet is the property of Air Liquide Medical Systems S.r.l. and no part of it may be reproduced without the manufacturer's written permission. All rights reserved.

TECHNICAL UPDATES

Air Liquide Medical Systems S.r.l. periodically reviews and modifies all its medical devices in order to improve their performance, safety and reliability. The instruction booklets are updated to ensure that they continue to be consistent with the characteristics of the devices marketed. If the booklet accompanying the apparatus is mislaid, a replacement copy can be obtained from the Manufacturer by quoting the data shown on the label.

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