Almohadilla AirTouch™ F20

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1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

See ResMed.com for other ResMed locations worldwide. AirFit and AirTouch are trademarks and/or registered trademarks of the ResMed family of companies.

For patent and other intellectual property information, see ResMed.com/ip.

© 2020 ResMed. 638237/1 2020-07

ResMed.com

638237

0123

ENGLISH

The AirTouch™ F20 foam cushion is compatible with the AirFit™ F20 mask system. When assembling and disassembling the foam cushion to the AirFit F20 mask, follow the steps in the AirFit F20 user guide. For normal use and maintenance of components other than the foam cushion, refer to the AirFit F20 user guide. In addition, follow the steps below for maintenance, troubleshooting and specific warnings related to use of the AirTouch F20 foam cushion.

AirTouch™ F20 cushion

The cushion is intended for single-patient reuse in both home and hospital/institutional environments and recommended to be replaced monthly.

⚠ CAUTION

- The cushion must not be submerged in liquid as this may damage the cushion and affect mask performance.**

- Discard the cushion if it is submerged in liquid or heavily soiled.**

Cleaning your mask at home

Daily/After each use:

If required, wipe your cushion with a CPAP wipe or equivalent and then allow to dry. Important – do not get cushion wet.

Monthly:

Replace the cushion by following relevant steps of the disassembly / reassembly instructions.

Troubleshooting

Mask leaks around the face.

Cushion is nearing the end of its life.

The cushion is designed to be replaced on a monthly basis. Cushion performance may deteriorate as it nears the end of its life. Adjust your headgear and if the problem persists, replace the cushion.

Cushioned is not positioned correctly.

Reposition mask on your face. If leak continues, turn off device, resettle your cushion and then turn device back on.

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